

Product Descriptions

The LIVILITI PAPTIZER Smart Sanitizer. The first portable smart sanitizer to integrate UV-C Light, Heat, and Circulated Air to provide 3 unique modes of sanitizing. The PAPTIZER reproduces UV-C light in the range of 260 nm to 280 nm using 40 high power LEDs. It can also reduce unpleasant smells by decomposing the molecular structure of the odor. In as little as 3 min you can kill over 99% of bacteria at the press of one button. The most advanced ozone and mercury free sanitizer in the market.

Ozone Free and Mercury Free

The LiViliti Paptizer Smart Sanitizer is the only sanitizer that is Mercury and Ozone Free. No downtime when your product is sanitized. Simply remove from the unit and your product is ready for use. Multiple research studies and reports show when biological organisms are exposed to deep UV light in the range of 200nm to 290nm it is absorbed by DNA, RNA and proteins. Absorption by proteins leads to the rupture of cell walls and death of the organism. Absorption by DNA or RNA is known to cause inactivation of the DNA or RNA double helix strands through the formation of thymine dimers. When enough of these dimers are created the DNA replication process is interrupted and the cell cannot replicate. There are NO known UVC resistant germs.

Sanitize your CPAP Supplies and Everyday High Touch Items

With the largest internal tray capacity in the market, you can clean anything that fits inside. Cpap masks, cpap headgear, humidifier chambers, surgical masks, cell phones, baby bottles, pacifiers, teething rings, hearing aids, dentures, makeup brushes, glasses, car keys, jewelry, wallet, credit cards, mail, etc.

Features:

- Industry leading quick 3 min disinfection time
- Easy to use 3 mode single function button settings with LED Display.
- Auto shut off when the product completes its cycle.
- Items are immediately ready for use after cleaning with no downtime.
- UV light, Heat and Circulated Air modes to provide the highest level of sanitizing.
- Sanitized Storage allowing you to secure a product for up to 24 hours.
- 10,000 hour long lasting UCV LEDs
- 2-year warranty

Benefits:

- Fast, reliable, and proven technology
- Safe and easy for anyone to use.
- No ozone, No mercury, No Voided Warranties
- No filters, costly adapters or maintenance required.
- Can disinfect a multitude of everyday items.
- Compact, portable, and affordable

What's Included:

- Liviliti Health Products Paptizer Smart Sanitizer
- Stainless Steel mounting rack
- Power adapter
- User Manual
- SGS Test Report
- 2 Year Warranty

Replacement Parts:

- Power Adapter (T5LPA)
- Stainless Rack (T5LSR)

Specs

Manufacturer	Liviliti Health Products
SKU	T5L
Product Dimensions	11.89 in L x 10.59 in W x 6.06 in H
Tray Dimensions	9.45 in L x 7.68 in W x 4.72 in H
Product Weight	4.29 lbs
Cleaning Method	UV-C LED
UV-C Wavelength	260-280nm
UV-C LED Quantity	40 pieces
Voltage	110 VAC (60Hz)
Operating Temperature	32F to 104F
Humidity	5% ~ 95%
Warranty	2 years

CPAP Sanitizer FAQ's

Q: Does Medicare or Private insurance cover CPAP Sanitizers?

A: No insurance will not cover it as they see it as a luxury item for consumers

Q: Can I stop washing my equipment if I use a sanitizer?

A: No. Continue to follow the manufacturer's directions for cleaning your product. Sanitizers do not remove dead skin, oil, makeup, or other organic materials.

Q: If I wash my equipment does it have to be completely dry before sanitizing?

A: It is recommended for the greatest results for the equipment to be dry.

Q: Is there a smell from using sanitizer?

A: Yes, there can be a slight smell to using any sanitizing product. To reduce and eliminate any smells make sure the mask is cleaned of any debris with soap and water before sanitizing.

Q: Can Ozone treatments affect a manufacturer's warranty of your CPAP device?

A: Some CPAP manufacturers are now excluding warranty coverage if a CPAP has been used with an Ozone device.